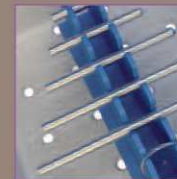
Swing Technologies, a THF bio-science division

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*The instrument is designed for uterus manipulation. This handling allows to reach in complete safety the posterior cul-de-sac and protects the surrounding vital organs (ureters, bladder, rectum...).*

*Moreover, the particular shape of its bodies allows to avoid carbonic gas leakage, differentiating it from other uterine manipulators.*

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

*The use of the Hystero-swing® widened to other interventions: supra-cervical hysterectomy, prolapsus, colpotomies, cervix cancers, endometriosis of the recto-vaginal wall...*

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*Picture 1 : Hystero-swing® assembled with the articulated arm known as "third hand" (ref. HYBA01 optional)*

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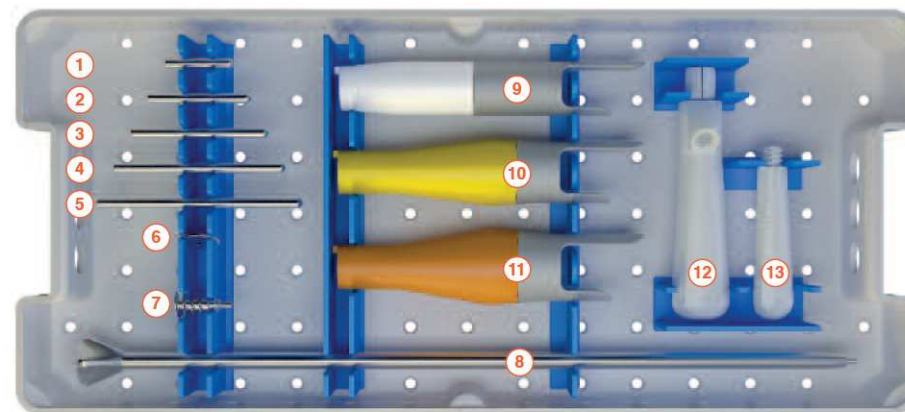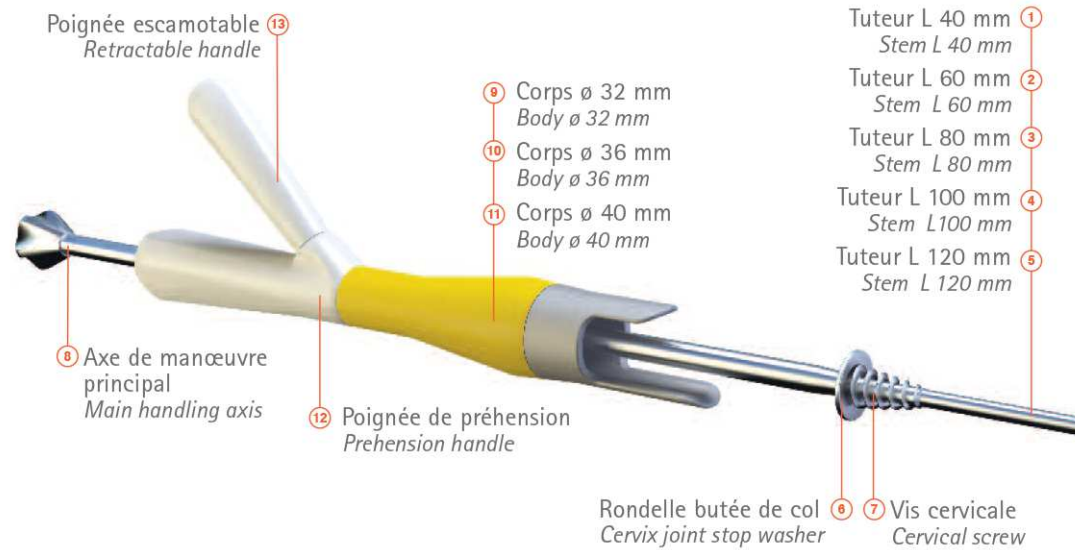*The assembly of the articulated arm is detailed in the Hystero-swing® video.*

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*The Hystero-swing® is intended to be general-purpose and can be adapted to the uterine anatomy.*

*This is why, before any operation, it is necessary to make two measurements for an appropriate use.*

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*The selection of the body's diameter depends on the vaginal gauge.*

*This calibration is carried out with a speculum which allows to visualize the vaginal diameter and to deduce the use of one of the three available sizes (Ø32-36-40 mm).*

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

*The selection of the stem length is made after having used a graduated hysterometer.*

*Before introducing the endo-uterine stem, it is necessary to expand the uterus with a 6 mm Hegar's bougie.*

*In order to choose the length of the stem, you must subtract the length of the cervical screw (30 mm) and the length of a 10 mm safety margin from the hysterometry that was measured before.*

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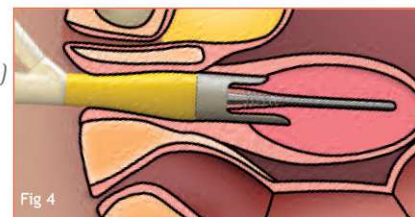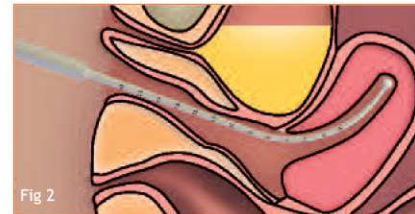**Table 1 : length calculation of the stem according to the hysterometry**

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*NB: these values are significant but need to be adjusted according to the way of each user will carry out the hysterometries. The stem selection will be adjusted according to the experience of each one.*

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*For all the indications of total hysterectomy, and moreover for complicated cases:*

- Douglas occlusions resulting of pelvi-peritonitis after-effects/sequelae,*
- Cesarean section antecedents,*
- Wertheim intervention.*

*The Hystero-swing® will enable a wide section of the utero sacral ligaments and will protect the vaginal barrel by grinding it, allowing best visualization of the vaginal flange if necessary.*

*Moreover, for the finalization of the section and hemostasis of the lateral cervical ligaments near the ureters (crucial and risky step), putting the dissector or scissors between the notches of the hystero-swing® makes it possible to divide the vagina from the inside*

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**NOTES :** In all the cases, the instrument is assembled with cervical screw (Fig 6) except in case of cancer for which you must use a uterine cervix stop joint washer without cervical screw (Fig 5) not to create histological damages

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*When the cervix is gaping, we must use the cervical screw and the cervix stop joint washer (Fig 7) to avoid the risks of uterine perforation.*

*It is recommended to choose an adapted stem length to cavity measurement (see table 1).*

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### PROLAPSE (*Gynecological and digestive surgery*)

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*Isolated rectoceles, complicated prolapsus with cystoceles, rectoceles and elythroceles.*

*The complete Hystero-swing® is assembled, without the cervical screw but with cervix washer (Fig 5), to present the retro-uterine cup for the dissection of the peritoneum opening the levator ani space.*

*Moreover, pushing the Hystero-swing® unfolds the prolapsed vagina, which ensures an easy adjustment of the promonto-fixation strip.*

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
*The complete Hystero-swing® is assembled without the cervical screw but with cervix washer (Fig 5).*

*This configuration allows the presentation of the posterior vaginal wall, which authorizes the safe section when we follow the concavity below the utero-sacral ligaments (checking the tip by under tissues palpation).*

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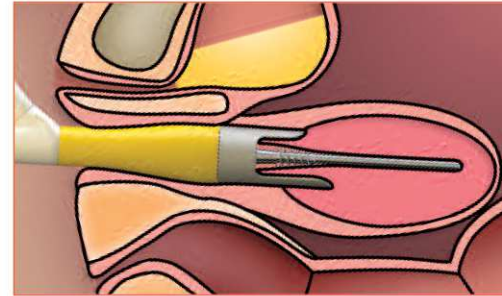*Fig 8: The vaginal tightness is continuously obtained by the body and allows the extraction of the surgical parts.*

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*This surgery is ensured by the same way as previously described, by an easy dissection on the vaginal barrel which fits perfectly the Hystero-swing®.*

*Please note that, in both cases, for the uterine conservation, the complete Hystero-swing® is assembled, without cervical screw, but with the cervix stop joint washer (Fig 5).*

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*The Hystero-swing® is assembled without cervical screw and cervix washer (Fig 5). Don't forget to withdraw Hystero-swing® before section with diathermy knife.*

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**IMPORTANT !** *General recommendations for maintenance, cleaning and sterilization are listed in the instructions for use (ref. ANCREV) according to DGS/R13/2011/449.*

Swing Technologies®, a THT bio-science® division

Balmer Médical SA > Rte de Provence > 1426 Concise >  
info@balmermedical.ch > Tél. +41 24 436 29 07 >  
[www.balmermedical.ch](http://www.balmermedical.ch)

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